



COURTNEY THORNE
nurse call systems

CONNECT WEAR

User guide



ACTIVATE THE CONNECT WEAR

To activate the Connect Wear, press and hold the red button (approx. 7 seconds) until the LED lights up, then quickly release and press the button once (within 1 second).

Once activated the Connect Wear is ready to use.

MAKE A CALL

To make a call, press the red button. The red LED will flash and the number which identifies the Connect Wear will appear on Display Panels.




If a call is not displayed it may be the Display Panel is not set up to receive the call.

RESET A CALL

To reset the call on the Connect Wear press and hold the red button (about 2 seconds) until the LED goes solid, then quickly release and press the button before the LED starts flashing (within 1 second).

CHANGING THE BATTERIES

The Connect Wear uses 1 x CR2430 coin cell battery, available to purchase from Courtney Thorne. To replace the battery, remove the strap and using a small Phillips screw driver, remove the 4 screws holding the back plate in place. Once the screws are removed, carefully lever the back cover off. Remove the board from the case and slide the battery from its holder taking care not to bend the pins. Fit the new battery (+ plus side up) and replace the board in the case. Replace the back cover, making sure not to damage the rubber seal.

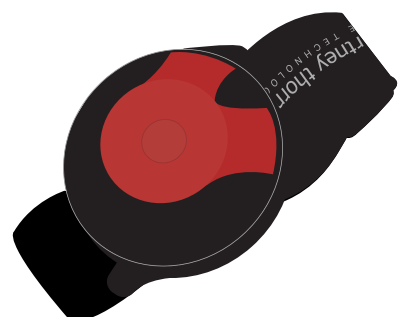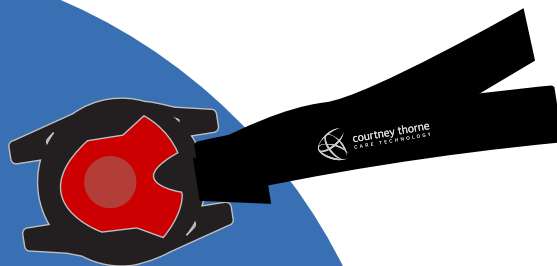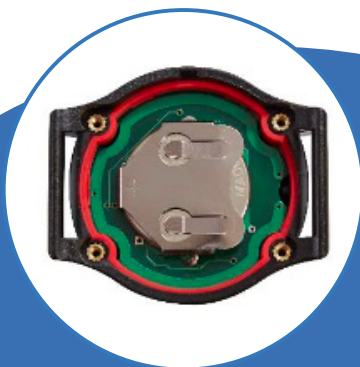



Courtney Thorne
C-T Centre, Elliott Road
Bournemouth BH11 8JS UK

www.c-t.co.uk
info@c-t.co.uk
+44(0)1202 573300



COURTNEY THORNE
care technology

